



Miracle

Stay New, Evermore



MIRACLE

With advanced collagenesis at its core,
MIRACLE helps your skin defy time, ensuring you always stay new.

Miracle is the foundation of Dexlevo's meso-ampoule line,
offering essential anti-aging benefits through collagenesis.

Miracle L and Miracle H focus on lifting and tightening sagging
skin, delivering long-term rejuvenation and improved elasticity,
with MIRACLE H also offering an option for those seeking an
immediate boost of hydration.




COMPOSITION

Miracle L and H stimulate fibroblasts to regenerate collagen using liquid PCL, with final results visible in 2 to 4 weeks. Miracle H also provides an immediate hydration effect for about a week after the procedure, thanks to hyaluronic acid.


Miracle L is composed of liquid type PCL



Miracle H is composed of liquid type PCL and HA

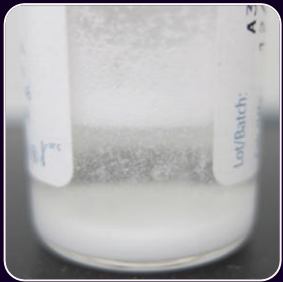

Polymer microparticle+carrier

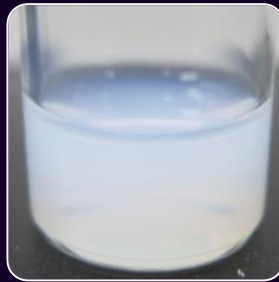

Fully Solubilized PCL

CESABP Technology

Collagenesis-Enabled Solubilized Active and Biodegradable Polymer

PCL has long been used in aesthetic treatments but faced limitations as a filler due to its large particle size, which made it difficult to use safely and effectively. However, with the development of solubilized liquid PCL, it is now possible to achieve both collagen regeneration and enhanced safety.



KEY FEATURE

1 Anti-Aging Care by Neocollagenesis

Miracle L and H stimulate fibroblasts to regenerate collagen, which diminishes with age and leads to sagging skin, ultimately restoring skin health and vitality.



1 Lifting and Tightening

By boosting collagen production, Miracle L and H fortify the skin's structure, effectively lifting and tightening for a firmer, more youthful appearance.

1 Moisturization

Miracle H provides immediate hydration with hyaluronic acid, ensuring the skin remains moisturized until collagen production begins, offering a seamless transition to long-term skin revitalization.

PROTOCOL

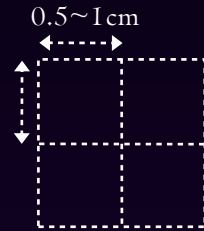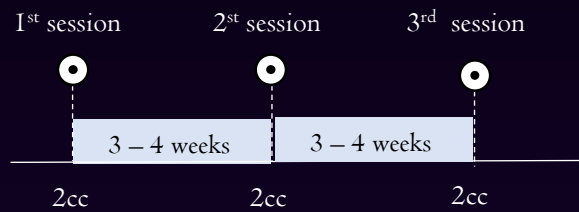

1. Cleanse well in lukewarm water.
2. Apply Lidocaine cream to target area.
3. Leave about 10 minutes.
4. Cleanse cream with alcohol cotton.
5. Apply 0.02cc ~ 0.05cc each point evenly to skin.
6. Carefully cleanse skin with alcohol cotton.
7. Evenly apply post-treatment cream to application area.



B&A CASE

Miracle H – 2 sessions at 4 weeks interval



BEFORE



AFTER

Miracle L – 3 sessions at 3 weeks interval



BEFORE



AFTER

Miracle L – 3 sessions at 4 weeks interval



BEFORE



AFTER

THANK YOU

